

Taima Rollator

Product Code: C4507-A-FX, C4505-D-FX

Specifications	C4507-A-FX	C4505-D-FX
Height	30" - 34"	31" - 37"
Height folded	30"	32"
Seat height	20"	23"
Seat width	15"	17"
Length	24"	27"
Width	23"	25"
Width folded	9"	9"
Max. user weight	330 lbs	330 lbs
Weight	13.7 lbs	13.9 lbs
Tire dimensions	8" x 1.4"	8" x 1.4"
Handle adjustment	5 x 1"	5 x 1.6"
Max. load shopping bag	10 lbs	10 lbs



Peak Care Equipment
149 Princes Highway
Unanderra NSW 2526
Tel: +612 4272 2688
www.peak-care.com.au

Distributor:

User Manual

Thank You!

The TAIiMA rollator is exceptionally light, strong and can easily be folded and stowed away. It provides ideal support during all your daily ventures as well a seat to rest. Before using the rollator for the first time please read this manual carefully and be sure to familiarize yourself with all the features of this product. The user should keep this manual available at all times and keep it with the product if it is passed on to a third party.

The manufacturer reserves the right to make changes to the information provided in these instructions as a result of technical developments and improvements.

This document may not be reprinted, translated or duplicated in any form whatsoever, in part or in full, without the manufacturer's prior written consent. There is no update service for this users manual. Please contact the manufacturer to keep abreast of new developments.

Safety information

The TAIiMA rollator is intended as a walking aid for persons with impaired mobility who weigh up to 330 lbs. It also provides a limited seating function for temporary rest. The rollator should be used only on firm, flat surfaces. It is not intended to be used for any other purpose - in particular as a climbing aid or to carry heavy loads. Any such use can be dangerous!

- Do not hang bags on the handles as this could cause the rollator to tip over.
- Use the service brake (by pulling the brake lever up) on inclines.
- Only use the rollator on firm, flat surfaces.
- Do not lean too heavily on the handles. Otherwise, the Rollator could tip over backwards.
- Be careful not to trap your fingers or clothes when folding up the rollator.
- Before using your rollator, make sure the locking pin is correctly engaged and the rollator cannot accidentally fold up.
- Upholstered parts become hot when exposed to direct sunlight and can cause damage to the skin if you touch them. These parts should be covered over. Alternatively, keep your rollator away from direct sunlight.

11. Cleaning

- Clean and disinfect the rollator at regular intervals. To do so, use mild soapy water with no scouring additives.
- You can disinfect the rollator using commercially available products.
- Never use steam degreasers. Never use scouring or caustic cleaning agents.

12. Maintenance

- The TAIiMA rollator requires no maintenance.
- If the braking function weakens, wheel movement becomes stiff or you notice any other functional impairment, please take the rollator to your dealer for service.

13. Reuse by Third Parties


- If you pass on your rollator to a new user, please remember to hand over all the technical documentation needed to ensure safe operation.
- In accordance with specifications, the rollator must first be inspected, cleaned and returned in pristine condition by a specialist dealer.

14. Warranty

- The TAIiMA rollator has a five-year warranty on the aluminum frame and a 12-month warranty on all other parts.
- Parts subject to wear are excluded from warranty.
- Warranty does not cover any sort of misuse of your rollator


9. Braking

- To reduce speed while walking just pull both levers evenly and gently upwards. Always apply the same amount of power on both brake levers.
- To lock the brakes, push the levers downwards. The brake levers will lock audibly into place. To release the brakes just pull the lever upwards again.

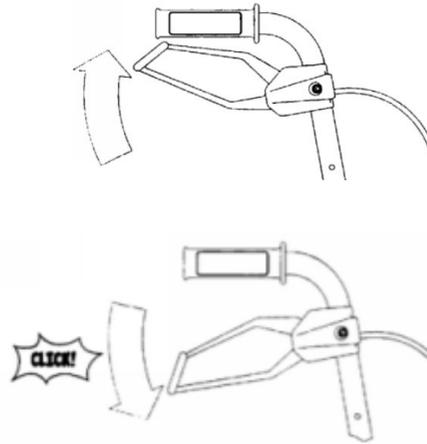


10. Curb Climbing

Climbing curbs easier you can lift the front wheels by pushing the rubber cap at the very end of the rollator frame downwards with your foot. Make sure you stand securely before using the curb climber.

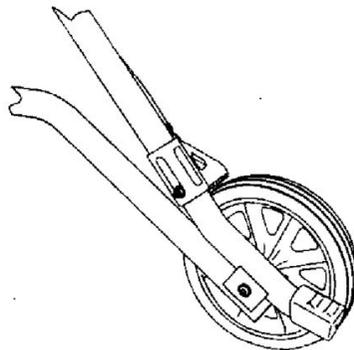

1. Rollator Parts

1. Backrest
2. Ergonomic handgrips
3. Brake lever
4. Cane holder
5. Height adjustment screw
6. Locking pin
7. Reflectors
8. Brake
9. Curb climber

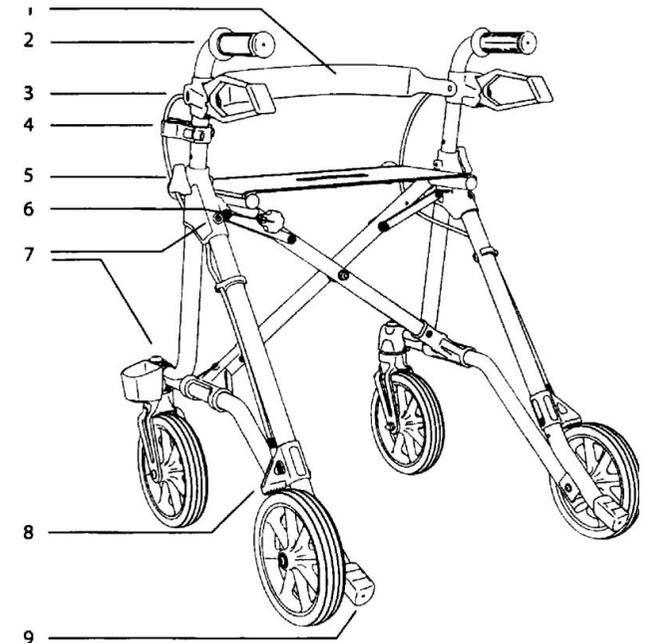


2. Contents

- Rollator TAiMA
- Backrest
- Cane holder L/R use
- Shopping bag
- User manual

3. Setting up/ Unfolding the Rollator


- Place the rollator in front of you in the direction of travel.
- Using both hands, press the tubes on either side of the seat down to the left and right until the locking pin (A) clicks into place.

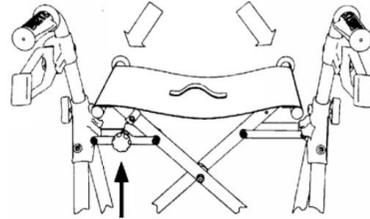

Caution! The rollator must never be used unless the locking pin (A) is properly engaged.


4. Folding the Rollator

Release the locking pin (A) by pulling it slightly backwards and pull up the seat by its folding strap. The rollator will now easily fold up.

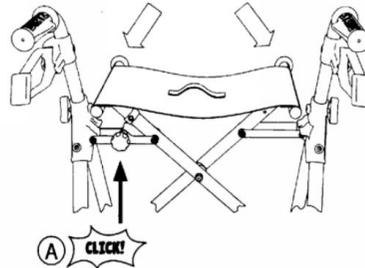

5. Handle Adjustment


Loosen screw (B) and remove it completely. Adjust the handles to the desired height. Place the screw (B) back in its hole and tighten it

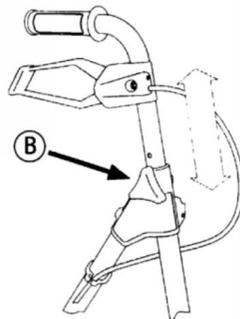

6. Brake Adjustment

The brake is set up correctly if there is no play in either the brake lever or the brake cable and if the wheels turn without any rubbing or grating.

The braking power can be adjusted using the set screw (C) of the brake cable on the brake blocks of the back wheels.

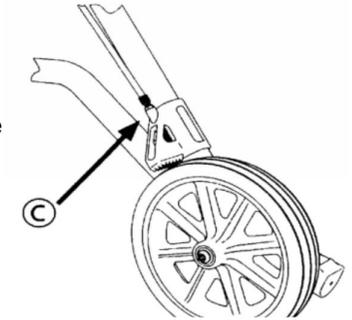

7. Walking with Your Rollator

- With the brakes set, position yourself inside the rollator. Grab the handgrips with both hands. As soon as you stand safely, you can loosen the parking brakes.
- Do not push the rollator too far in front of you. In order to give good support, the rollator should be kept close to your body.
- Control your speed by gently applying both brakes evenly.
- Walk upright and look forward to be able to avoid any upcoming obstacles early. Walking backwards with your rollator is difficult, and not recommended.

8. Reuse by third parties:

If you pass on your rollator to a new user, please remember to hand over all the technical documentation needed to ensure safe operation. In accordance with specifications, the rollator must first be inspected, cleared and returned in pristine condition by a specialist dealer.