

# Thank You!

Thank you for purchasing the Peak Toilet Seat Raiser

Peak Toilet Seat Raisers are designed to assist with Toilet transfers, are available with or without Lids.

Kindly take time to go through this manual which gives you more details about your Toilet Seat Raiser. Kindly keep this booklet handy and use it as and when required.

Peak Toilet Seat Raiser is warranted to be free of defects in material and workmanship for twelve months from the date of purchase. This device was built to exacting standards and carefully inspected prior to packing and shipping. This warranty is an expression of our confidence in the materials, our workmanship and an assurance of the same to our customers. In the event of a defect covered by this warranty, we will at our option supply parts or replace the unit.

This warranty does not cover device failure due to owner's misuse or negligence or normal wear and tear. This warranty does not extend to nondurable components such as clips which are subject to normal wear and need periodic replacement.

If you have any queries, please contact the distributor.

**Peak Care Equipment**  
149 Princes Highway  
Unanderra NSW 2526  
Tel: +612 4272 2688  
[www.peak-care.com.au](http://www.peak-care.com.au)

**Distributor:**

## Peak Toilet Seat Raiser 50Mm With Lid & Clips

Product Codes: 9346376020107, 9346376020114, 9346376020121



**User Manual**

Staple Receipt Here:

## INSTALLATION

- Remove the existing toilet seat from the bowl. Seats are generally attached by a pair of bolts/nuts located under the rear rim of the toilet.
- Unscrew the side knobs until they extend out from the Raised Toilet Seat.
- Position the Toilet Seat Raiser onto your toilet bowl.
- Tighten the side knobs until the fixing is secure. Do not over tighten. You should not be able to move the Raised Toilet Seat, once fitted.

## REGULAR MAINTENANCE

- Regularly check the side knobs are tight and the toilet seat raiser is firmly clamped to the toilet bowl.
- Regularly check the Toilet Seat Raiser for damage and wear, paying special attention to the seat and the side knobs. Do not use the Toilet Seat Raiser if the plastic seat is cracked or deformed in anyway.
- The seat can be removed fully for thorough cleaning by unscrewing the side knobs.
- To clean the unit after each use, wipe clean as required with a bathroom disinfectant and a soft cloth.
- Do not use any abrasive cleaners or scourers that will scratch.
- Do not use water temperature in excess of 80 Degrees

## SAFETY INSTRUCTIONS

- Maximum load capacity is 136 Kg (300lbs).
- The load capacity is not based on the lid of the toilet seat raiser. The lid is not suitable for use as a seat.
- The toilet seat raiser is not suitable for continuous use as a seat.
- Do not hold onto the seat as a support when getting on and off the toilet seat raiser.
- Under no circumstances should you stand on the raised toilet seat.
- Ensure that the side knobs are tight enough to hold the seat to the toilet bowl.

## AVAILABILITY

- 2" (50mm), 4" (100mm) and 6"(150mm) with lids
- 2" (50mm) and 4" (100mm) without lids