

Stocking Slider

Description:

The stocking slider is used to aid the patient and/or carer in putting on tight fitting open-toe compression stockings and can be used on the arms or legs. The use of this product helps to prevent tearing of the patient's skin and protect existing skin lesions.

It is made from strong, slippery nylon with a reinforced webbed loop at one end to help remove the stocking sliders from under the applied bandage.

Hand or machine wash up to 80°C. Do not tumble dry – air dry only.



Dimensions:

Length: 34cm (when tucked in on itself)

Instructions for use:

To use:

1. Make sure stocking slider is tucked inside itself (to form a cone shape) with the webbing loop exposed on the outside.
2. Insert foot, ankle and shin (or arm) into the stocking slider.
3. Slide the compression stocking over the stocking slider.
4. When the stocking is in place, grip the webbing loop with your hand and pull the stocking slider away from the foot until completely removed.



The aid is a Class 1 Medical Device (non-invasive and non-sterile).