


# Torkel – toilet paper tongs

Torkel toilet paper tongs provide a functional grip and extended reach to handle intimate hygiene.


## Easy to handle

When grip and mobility function are restricted, these tongs can be used for toilet paper, wash cloths and small towels. They are made of a material that can withstand boiling.

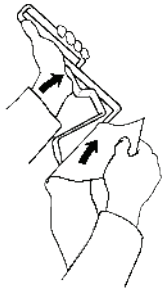


## How to use Torkel toilet paper tongs:


**1.**  
Tear off a piece of toilet paper (approx. 400 mm).  
Open the tongs by squeezing the handle, then thread the paper between the tongs.

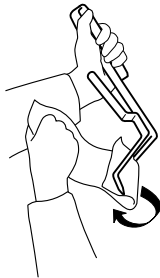

**2.**  
Roll the paper loosely around the tongs so that it covers the tip.

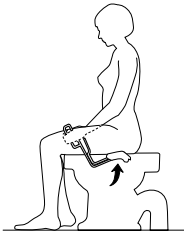

**3.**  
Place the tongs between the legs and wipe.

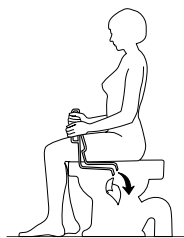

**4.**  
Open the tongs and shake to drop the used toilet paper into the toilet. Rinse the tongs.



**Description**  
Torkel Toilet Paper Tongs

**Length**  
330 mm

**Weight**  
100 g

**Material**  
Polyamide plastic

**Order Code**  
80304001



max 85°C  
max 185°F  
for maximum of 3 minutes



Clean the product with a solvent-free cleaning agent (pH level 6–9), or with a 70% disinfectant solution. Can be decontaminated at max. 100°C / 212°F (boiling) if so required.