

# Immedia Transfer Boards

## A WIDE RANGE OF SOLUTIONS FOR SEATED TRANSFERS

The transfer board is intended to bridge shorter gaps when performing seated transfers. The board's low friction upper surface allows the user to transfer independently or with assistance. Immedia Transfer Boards are used when transferring between bed and wheelchair / shower chair, between wheelchair / chair / toilet chair or to and from the car. All Transfer Boards have a non-slip surface on the underside. Combining a transfer board with a glide cushion provides even lower friction and is recommended when the transfer involves bare skin, for instance to and from a shower chair or toilet.



## Immedia Butterfly

Immedia Butterfly is a long, wide and stable transfer board. It has cut outs for positioning against the wheel of a wheelchair and non-slip pads on the underside which allows for good positioning and avoids sliding out of place.

### Immedia Butterfly GlideWing

Butterfly can be used with the special Butterfly GlideWing, which is mounted on the board. The GlideWing slides along the board and provides additional slide function.



## Immedia 3B-Board

Immedia 3B-Board is a stable transfer board with a traditional curved shape. The board can be placed either curve forward or curve backward depending on the transfer situation and surrounding obstacles, eg: the wheel of the wheelchair.

### RECOMMENDED USE!



1. Place the Immedia Butterfly close to the users seat, and slide the GlideWing under one seat bone.



2. If possible, the user holds on to the opposite armrest when she starts to slide towards the wheelchair. Use the Immedia Pediturn to ease turning.



3. When the transfer is complete, remove the Immedia Butterfly and engage wheelchair armrests and legrests.

### TIP!

When transferring with a board, ensure the surface you are moving to is slightly lower and use gravity to make the transfer easier.

## Immedia E-Board

Immedia E-Board is a thin and flexible transfer board that allows easier application and removal.



Immedia E-Board with wings.

### E-Board with wings

Immedia E-Board with wings has wings along the sides, which can be bent up or down. This provides more sturdiness and stability. The upward wing is used to protect against the wheel of the wheelchair. Available in two lengths.



Immedia E-Board without wings.

### E-Board Compact

A smaller and lighter board that is easily carried and suitable for the more active and independent user. Immedia E-Board Compact is available in two lengths.



ONLINE TUTORIAL!



### Learn more ...

... about how to use the transferboards. Our new animated film *Transferboards – E-Board* gives you useful instructions for your daily work.



## Immedia Dolphin

Dolphin is a sturdy and stable transfer board uniquely designed for transfer to the toilet. The board has cut-outs for positioning against the wheel of the wheelchair and non-slip pads, which allows good positioning and avoids sliding out of place. The cut-out allows the user to remain on the board while on the toilet.



Immedia Dolphin is uniquely designed for transfer to toilet.

### INTENDED USE

- Transfer from bed to wheelchair / shower chair
- Transfer in and out of the car
- Transfer between wheelchair, chair, toilet, shower chair, commode

#### DESCRIPTION

#### WIDTH X LENGTH X HEIGHT

#### ITEM NO.

#### SPECIFICATIONS

#### IMMEDIA

	CM	INCH	
~ 3B-Board, curved, white	23 x 77	9.1 x 30.3	IM403
~ E-Board Small, black	20 x 45 x 0,5	7.9 x 17.7 x 0.2	IM410
~ E-Board Medium, black	25 x 60 x 0,5	9.8 x 23.6 x 0.2	IM408
~ E-Board Wide, with wings black	33 x 60 x 0,5	13 x 23 x 0.2	IM404
~ E-Board Long, with wings, black	33 x 75 x 0,6	13 x 29.5 x 0.2	IM405
~ Butterfly	32 x 66	12.6 x 26	IM401
~ Butterfly GlideWing L	35 x 40	13.8 x 15.7	IM4001
~ Butterfly GlideWing XL	35 x 47.5	13.8 x 18.7	IM4002
~ Dolphin Toilet Board, white	Length: 69	27.2	IM409

#### Immedia E-board and 3B-board

MATERIAL: Polyethylene

#### Immedia Butterfly and Dolphin

MATERIAL: Fibreglass

#### Immedia Butterfly GlideWing

MATERIAL OUTSIDE & INSIDE: Nylon

PADDING: Polyester

#### WASHING INSTRUCTIONS

TRANSFER BOARDS: Clean with soapy water or disinfectant.



BUTTERFLY GLIDEWING: Wipe off with soapy water or disinfectant. Do not use fabric softener or bleach.

