

Hollywog® WiTouch™ Pro

The worlds first true wireless
TENS for back pain



Ready to use; no programs to set or learn
Portable and convenient for travel
Easy to use and self apply


Pack includes

- Remote Control
- 5 pairs reusable Gel Pads
- Batteries (2 x AAA;
1 x lithium coin CR2032)
- Quick Start Guide & User Manual
- Storage case

WiTouch™ Pro is the first wireless
remote controlled pain relief device
incorporating TENS technology to
specifically target back pain.

The thin and flexible design perfectly contours the back for maximum surface contact. Maximises energy use providing over 150, 30-minute treatment sessions per battery life. Innovative device is safe, drug-free, easy to use, discrete and comfortable to wear.

- 110mA output – 20% higher than the majority of portable TENS devices
- 2 times larger treatment area for more effective back pain relief

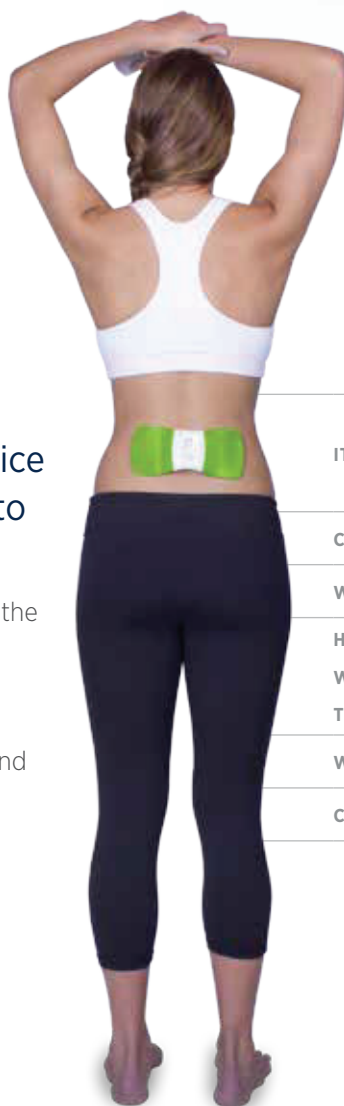

ITEM	WiTouch Pro Kit	Replacement reusable Gel Pads (pack of 5 pairs)
CODE	11-1500i	11-1500
WARRANTY	1 year	
HEIGHT	90 mm	
WIDTH	191 mm	
THICKNESS	18 mm	
WEIGHT	136 g with batteries	
CONFORMS TO	ARTG 203804	



Phone 1300 734 862
support@patienthandling.com.au

Hollywog® WiTouch™ Pro

The worlds first true wireless
TENS for back pain

*Thin, lightweight and flexible design contours the
back for a perfect fit on almost any body shape*

*Comfortable & discrete; wear
under your clothing*

*No wires, bulky back
wraps or belts*



Transcutaneous Electrical Nerve Stimulation (TENS) delivers tiny electrical impulses to the nerve fibres that lie underneath the skin's surface, providing pain relief by blocking pain signals to the brain via the spinal cord and peripheral nervous system.

TENS is used for the symptomatic relief and management of chronic intractable pain and adjunctive treatment of post-surgical or post-traumatic acute pain.

TENS can be an effective way to reduce or eliminate pain, however may not be suitable for all patients.

Certain daily activities or medical conditions can contraindicate, or make inadvisable, the use of a TENS machine. TENS machines are not suitable for use with implanted pacemakers, defibrillators, infusion pumps and other such personal medical devices. Please consult your healthcare professional before use; and only use under their continued supervision.



reddot design award
best of the best 2013

GOOD
DESIGN
Australia



THERAPEUTIC DEVICES SHOULD ONLY BE USED IN ACCORDANCE WITH MANUFACTURERS INSTRUCTIONS AND UNDER THE CONSENT, SUPERVISION AND MANAGEMENT OF A SUITABLY QUALIFIED HEALTH PROFESSIONAL.



**PATIENT
HANDLING™**

Phone 1300 734 862
support@patienthandling.com.au